


Monidor remote monitoring, Quick guide

Basic use




Turn on the device by pressing the power button.

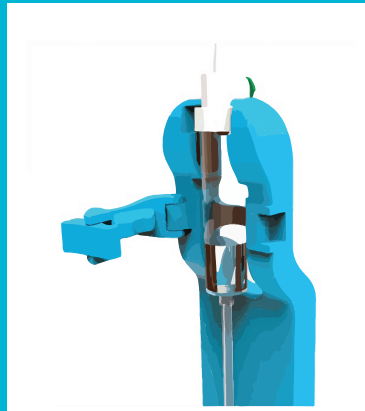

Attach the Monidrop device to the drip chamber.




Use the roller clamp or the flow regulator to set the infusion rate.

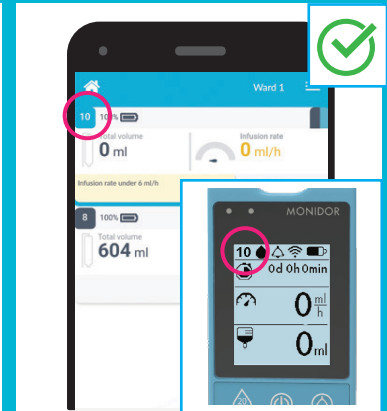


Use the device numbers to identify your Monidrop device and IV Screen application card.


Advanced settings and notifications

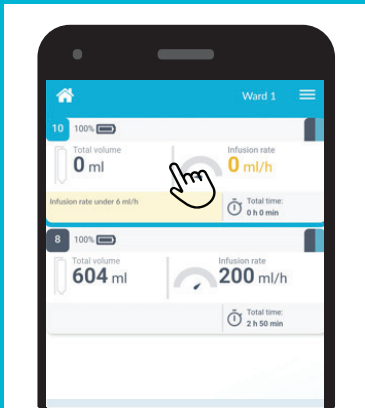


If you wish, you can make additional settings by clicking on the device card you are using.

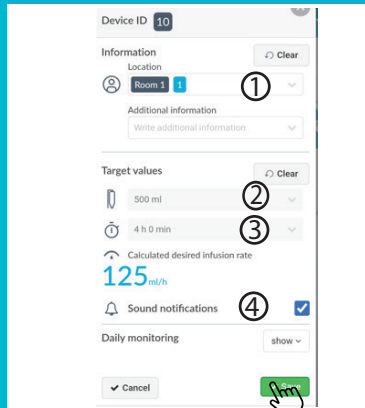

Set the patient location (1) and targets for liquid volume (2) and duration of treatment (3). Sound notifications are activated automatically as settings are set (4).

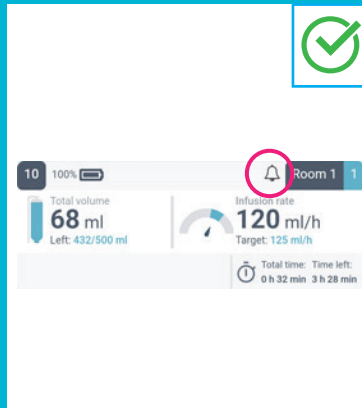

The treatment is progressing as planned, when the card is displayed in blue in the IV Screen application. The bell icon indicates that sound notifications are on.

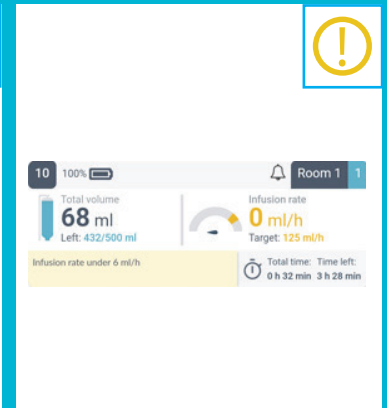

Infusion deviations are highlighted in yellow, and the reason for the notification is shown in text.